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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION AND DESCRIPTION OF THE DEVICE

The digital breathalyzer is a portable device for measuring alcohol content in exhaled air. It is equipped with an LCD display and several control buttons that allow for quick and precise readings of results.

The mouthpieceless breathalyzer is portable and easy to store, so you can always have it with you, no matter where you go. Now you can be sure that you'll always be ready to test your sobriety when the need arises.

ACTIVATION

The "warm up" symbol is displayed and a countdown from 10 to 00 begins, which means that after this time the tester is ready to use. The BLOW message will be displayed. Bring the sensor closer and blow air for 3-5 seconds. Read the test result from the breathalyzer LCD display.

Above a concentration of 0.02% the word CAUTION will be displayed.

Above 0.05% concentration the word DANGER will be displayed.

To get more accurate results, the manufacturer recommends checking 2-3 times.

Setting the clock

In real mode, press <ADJ to select 12 or 24 hour format.

Press and hold the <SET button for 2 seconds to enter the real-time hour setting. The hour digit on the LCD clock display will flash. Press the <AD button to adjust.

Press the SET> button again to enter the minute setting. The minute digits will flash. Press the <ADJ> button to set.

Press SET button to confirm

Alarm

In real-time mode, press the alarm node. MODE button to enter alarm mode.

Press the ADJ button to turn the alarm on or off.

Press and hold the <SET) button for 2 seconds to turn on the alarm.

timer hour setting, the hour digit flashes, press <ADJ> button to adjust.

Press the SET button again to enter the minute setting. If the minute digit is flashing, press the <ADJ button to set it.

Press the <SET button to confirm.

Thermometer

In real-time mode, press the <MODE> button twice to enter temperature mode.

Press the <ADJ> button to select C or F format.

Countdown time

press the MODE button 3 times to enter the setting. Press and hold the SET button for 2 seconds to enter the timer hour. Press the (ADJ) button to adjust the setting. The hour digit will be displayed.

Press the SET button again to enter the minute setting. The minute digit will flash. Press the <ADI button to adjust

Press the SET button to confirm.

Press <AD button to start countdown, when the timer value is not 00:00, press <ADI button again to stop the timer.

When the countdown reaches 00:00, an alarm will sound and then the counter value will be reloaded.

BATTERY REPLACEMENT INSTRUCTIONS

Please insert the batteries correctly, paying attention to the +/- polarity.

Only use one type of battery.

Always replace the entire set of batteries.

Exhausted batteries must be removed from the device and never thrown into fire.

Never attempt to charge regular batteries.

Store unused batteries in their packaging away from metal objects.

Do not leave children unsupervised.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Displays in percentage and grams per liter
- Auto power off function
- Backlit LCD display
- Color: White
- The product is CE and RoHS certified
- Measurement range from 0 to 1.9 g/l
- Accuracy 0.01%
- Dimensions 12cm x 4cm x 2.5cm

- Power supply: 2 x AAA battery
- Sensor: Advanced semiconductor alcohol oxide sensor
- Accuracy: -0.01%-0.02% at 0.05% BAC
- Input Power: 2 x 1.5V “AAA” Alkaline Batteries Operating Time: Warm-up: 10 sec, Response: <5 sec
- Auto power off: about 1 minute

SAFETY INSTRUCTIONS

- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Do not use any stimulants such as paints, insecticides, or alcohol near the tester.
- Do not place the tester in a closed room with polluted air.
- Each person's body reacts differently to alcohol.
- Never immerse the device in water.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- The product is not intended for use by children.
- Make sure that packaging materials are not left unattended. Children may start playing with them, which is dangerous.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - does not function properly, - has been stored for an extended period under unfavourable conditions or - has been excessively stressed during transport.
- It is forbidden to use the product if any of its parts are damaged. In the event of damage to the cable, it is forbidden to make repairs yourself.
- Do not disassemble the device yourself.
- Do not use old and new batteries, different brands or chemicals.
- Do not charge, damage or open the battery.
- Do not swallow the battery. If the battery is swallowed, consult a doctor immediately.
- Always use the product as intended.

BATTERIES/ACCUMULATORS SAFETY INSTRUCTIONS

- Batteries/rechargeable batteries should be kept out of reach of children. If swallowed, consult a doctor immediately!
- Disposable batteries must not be recharged. Batteries/accumulators must not be short-circuited and/or opened. This may cause overheating, fire or explosion.
- Never throw batteries/accumulators into fire or water.
- Batteries/accumulators should never be subjected to mechanical stress.
- Risk of acid leakage from batteries/accumulators.
- Avoid extreme conditions and temperatures that may affect batteries / accumulators, e.g. radiators / direct sunlight.
- If batteries/accumulators leak, avoid contact of skin, eyes and mucous membranes with chemicals! Immediately rinse affected areas with clean water and seek medical attention!
- Leaking or damaged batteries/rechargeable batteries can cause chemical burns when in contact with skin. Therefore, suitable protective gloves should be worn in such cases.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT

Used electrical and electronic products, due to environmental protection, must not be thrown into household waste with regular municipal waste, but disposed of properly. Information on collection points and their opening hours is provided by the relevant office.

This product complies with the requirements of the relevant European and national directives that apply to it. The product meets European and national requirements on the safety of devices and products.

This product complies with the requirements of the relevant European and national directives that apply to it, RoHS.

Improper disposal of batteries/accumulators poses a risk to the environment!

Batteries/accumulators should not be disposed of with household waste. They may contain harmful heavy metals and should be treated as special waste. The chemical symbols for heavy metals are as follows: Cd = cadmium, Hg = mercury, Pb = lead.

Therefore, used batteries / accumulators should be taken to municipal hazardous waste collection points.

The built-in battery must not be disassembled for disposal. The product must be handed over in its entirety to a collection point for used electronics.

We reserve the right to make changes to the text, design and technical data of the product without notice.

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